Copran Color
(Copran Color EVO/ CopraSupreme EVO / CopraSmile EVO)
Basis UDI-DI: ++EWPDCCLKC

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| | |
|------------------------------|--|
| Manufacturer | Whitepeaks Dental Solutions GmbH Alfredstr. 81 - 45130 Essen – Germany SRN: DE-MF-000021120 |
| Product/ product type | Colouring liquid for zirconium dioxide ceramics |
| Product form | Colouring liquid in various shades - medical device class IIa |
| User group | Instructed users, who manufacture dental restorations from zirconium dioxide ceramics and colour these |
| Shelf life/ storage | 3 years at room temperature and in closed airtight containers. Store in original container. |

Indication/ intended use

The products of the Copran Color product group are exclusively suitable for coloring the respective Whitepeaks zirconium dioxide ceramics in dental technology.

| Product | Intended use |
|------------------|--|
| Copran ZRI EVO | Colouring liquid (dipping colour) for colouring white/ opaque zirconium dioxide ceramics before sintering |
| CopraSupreme EVO | Colouring liquid (dipping colour or paint on colour) for colouring translucent zirconium dioxide ceramics before sintering |
| CopraSmile EVO | Colouring liquid (paint on colour) for colouring translucent zirconium dioxide ceramics before sintering |

The products of the Copran Color product group are medical devices intended for the colouring of dental prostheses intended for long-term use, which are partially introduced into the human body by means of a clinical intervention, but in this case are inserted into the teeth or fixed to the tooth substance by means of cement and are thus assigned to Class IIa.

The type of coloured denture depends on the respective Whitepeaks zirconium dioxide ceramic of the Copran product group. Please refer to the instructions for use of the Copran product group.

Material properties/ technical data

The Copran Color colouring liquids are fluid, odourless, water-based, acid-free, free of oily substances, easy to apply, easy to store, easy to dispose of and have a long shelf-life.

Description

The products of the product group Copran Color are exclusively suitable for the colouring of zirconium dioxide ceramics in the dental technology. This colour adjustment has the advantage, that the dental restoration can be adjusted individually for the patient.

When used according to instructions Copran Color is completely biocompatible and non-toxic. Copran Color is water-based and does not contain acid therefore it does not corrode metal instruments or metal workbench surfaces. Also, it does not corrode the heating elements in the sintering furnace. Copran Color does not contain

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

Copran Color
(Copran Color EVO/ CopraSupreme EVO / CopraSmile EVO)
Basis UDI-DI: ++EWPDCCLKC

oily substances, makes full use of the capillary effect of the zirconium and infiltrates quickly and deeply into the restoration. At the cutting of sinter drops and connecting bars you will not experience any lighter areas or blotches. If a colour is too intensive for your taste, the colouring liquids can be easily dissolved with distilled water.

Instruction for use

The restoration should be completely finished and completely dry before colouring. The restoration should be processed dry.

Preparation

To achieve a perfect surface and maximum transparency, we recommend cleaning the restoration in the White-Sonic ultrasonic cleaning unit and distilled water. Please use only our recommended ultrasonic cleaning device. Devices from other companies could be too "strong" and damage the restoration. Swivel the restoration with plastic tweezers in the distilled water of the ultrasonic unit for 5 to 10 seconds until no more "dust cloud" comes off the restoration. Remove excess water from the restoration by blowing it off with oil-free air or by blotting it dry with cellulose or cotton swabs. Dry the restoration under an infrared lamp or using an oven. Before sintering, the restoration must be completely dry. Make sure that the water never boils during drying, as this can cause cracks in the zirconia. Take the required colouring liquid, make sure the container is properly closed and shake it for a few seconds to make sure there is no sediment.

Dipping technique

Now completely immerse the restoration in the colour liquid of your choice for approx. 45 seconds. Remove the restoration and remove the remaining liquid with a moistened brush, by blowing it off with oil-free air or by drying it with cellulose or a cotton swab. Close the container of paint tightly and put it back. If a restoration is too large for the original vessel, pour the colour liquid into a clean vessel and colour your restoration there. Afterwards, clean the vessel thoroughly with water and washing-up liquid.

Painting technique

Start the painting process with a brush. We recommend to apply the colouring liquids in the following painting sequence: Margin area, incisal areas, occlusal areas, tubercles and finally dentin colour.

Use the blocker liquid carefully and sparingly. It prevents other liquids from infiltrating into the restoration and therefore has to be applied before any other liquids. It leaves the area completely in the natural zirconium colour. After applying the effect colours, colour the rest of your restoration with the respective dentin liquid. You can also apply the dentin liquid from the inside of the crown to create the illusion of a dentin core.

With a little skill and training you will be able to create fantastic lively results.

Sintering

Dry your restoration completely as stated above. A not completely dry restoration may result in colour alteration or opacity of the zirconium dioxide. Sinter your restoration as described in the instructions for use of the utilised zirconium dioxide material.

After the final sintering the restoration can be ground, if necessary, in a wet grinding technique with diamond coated grinding tools. Sintered diamonds, korundum stones or hard metal burs must not be used.

Safety instructions

Warning: May contain cobalt (II) chloride hexahydrate. May cause cancer when inhaled. Suspected of causing genetic defects. May damage fertility. Always observe the following safety precautions

May contain nickel (II) chloride hexahydrate. May cause cancer when inhaled. Suspected of causing genetic defects. May damage fertility or the unborn child. Always observe the following safety precautions.

Always wear respiratory protection (filter class FFP2), tight-fitting safety goggles, protective gloves and protective clothing and switch on extraction equipment. Ensure good ventilation. Do not inhale vapor/aerosol. Avoid contact with skin, mouth, eyes and clothing. Do not eat or drink while working. Keep away from food and drink. Wash hands after use. Remove contaminated clothing and protective equipment before entering areas where food will be eaten. Protect from heat and sunlight. Keep away from sources of ignition. Do not smoke.

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

Copran Color
(Copran Color EVO/ CopraSupreme EVO / CopraSmile EVO)
Basis UDI-DI: ++EWPDCCLKC

Disposal

Dispose of product and packaging in accordance with local/ regional/ national/ international regulations. Do not dispose of together with household waste. Do not allow to enter water, ground water or sewage system.

Notice

Any serious incident, that has occurred in relation to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

The summary report on safety and clinical performance according to (EU) MDR 2017 / 745 Article 32 (SSCP) is available in the European Database for Medicinal Products (Eudamed) under the associated base UDI-DI as soon as the corresponding database module exists.

<https://ec.europa.eu/tools/eudamed>

Explanation of the markings on the packaging

Symbol for „article number“

Symbol for „LOT number“

Confirmation: The product complies with the applicable European directives.

Symbol for „follow the instructions for use“

Symbol for „is a medical device“

Symbol for „applicable until“

Rx only

Symbol for “Caution: US Federal law restricts this device to sale by or on the order of a licensed physician or dentist.”

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

Copran Color
(Copran Color EVO/ CopraSupreme EVO / CopraSmile EVO)
Basis UDI-DI: ++EWPDCCLKC

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